









# Hygiene plan

What	When	How	Product / Dosage / Exposure Time
<p>Hands</p> 	<p>before/after each customer or hearing aid contact</p> <p>when necessary / after finishing work</p>	<p>Removing of relevant pathogenic germs by rubbing with antimicrobial gel. Also for use without water and towel.</p> <p>Rub in for skin care.</p>	<p><i>Cedis</i> Profi ANTISEPT Rub 3 ml for 30 seconds</p> <p><i>Cedis</i> Profi CARE</p>
<p>Hearing Aid</p> 	<p>before/after each service</p>	<p>Spray with cleansing disinfectant, then dry thoroughly with a moisture absorbing cloth after exposure time.</p> <p>Wipe with cleansing disinfectant, then dry thoroughly with a moisture absorbing cloth after exposure time.</p> <p>Rub into the surfaces, then dry thoroughly with a moisture absorbing cloth after exposure time.</p>	<p><i>Cedis</i> cleansing spray 1 minute exposure time</p> <p><i>Cedis</i> cleansing wipes 1 minute exposure time</p> <p><i>Cedis</i> med WB stick and wipes 5 minutes exposure time</p>
<p>Earmold</p> 	<p>before/after each service</p>	<p>Disinfecting or cleaning in the immersion- or ultrasonic bath, then dry thoroughly with a moisture absorbing cloth after exposure time.</p> <p>Rub into the surfaces, then dry thoroughly with a moisture absorbing cloth after exposure time.</p>	<p><i>Cedis</i> Profi UltraActiv 7.5% / 1 minute exposure time</p> <p><i>Cedis</i> med WB stick and wipes 5 minutes exposure time</p>
<p>Impression</p> 	<p>after impression taking</p>	<p>Wipe with cleansing disinfectant, then dry thoroughly with a moisture absorbing cloth after exposure time.</p>	<p><i>Cedis</i> cleansing wipes 1 minute exposure time</p>
<p>Tools</p> 	<p>after each application</p>	<p>Spray with cleansing disinfectant, then dry thoroughly with a moisture absorbing cloth after exposure time.</p> <p>Wipe with cleansing disinfectant, then dry thoroughly with a moisture absorbing cloth after exposure time.</p> <p>Disinfecting or cleaning in the immersion- or ultrasonic bath, then dry thoroughly with a moisture absorbing cloth after exposure time.</p>	<p><i>Cedis</i> cleansing spray 1 minute exposure time</p> <p><i>Cedis</i> cleansing wipes 1 minute exposure time</p> <p><i>Cedis</i> Profi UltraActiv 7.5% / 1 minute exposure time</p>
<p>Workplace</p> 	<p>on visible contamination / after finishing work</p>	<p>Wipe with cleansing disinfectant, dry off after exposure time.</p>	<p><i>Cedis</i> Profi SURFACE 5 minutes exposure time</p>
<p>Earphones</p> 	<p>after each application</p>	<p>Wipe with cleansing disinfectant, then dry thoroughly with a moisture absorbing cloth after exposure time.</p>	<p><i>Cedis</i> cleansing wipes 1 minute exposure time</p>
<p>Otoscope Funnel</p> 	<p>after each application</p>	<p>Disinfecting or cleaning in the immersion- or ultrasonic bath, then dry thoroughly with a moisture absorbing cloth after exposure time.</p>	<p><i>Cedis</i> Profi UltraActiv 7.5% / 1 minute exposure time</p>

Please adhere to the product information.  
Please use disposable towels for drying.  
For further details about a. m. egger products please refer to our latest product catalog.

For further information please contact:  
egger Otoplastik + Labortechnik GmbH  
Aybühlweg 59  
87439 Kempten/Germany

Tel.: +49 831 58113-60  
Fax: +49 831 58113-14  
E-mail: sales@egger-labor.de  
Internet: www.egger-labor.de